



# DISINTEK™ GTA 1.5% TEST STRIPS

For CIDEX® or MetriCide® High Level Disinfectant with 1.5% MEC

**Product Code:** 5136A

**Description:** DISINTEK GTA 1.5% TEST STRIPS

**Packaging:** 2 bottles/kit; 50 strips/bottle

## DIRECTIONS FOR CIDEX® OR METRICIDE®:

1. Immerse indicator pad into **CIDEX** or **MetriCide** disinfectant solution and remove immediately.
2. Touch the long edge of the strip to a paper towel for three (3) seconds to remove excess sample.
3. Lay strip, with the pad facing up, on a paper towel.
4. **Ninety (90) seconds** after immersion, interpret and record results.

## INTERPRETATION OF RESULTS:

The color developed on the indicator pad indicates whether the concentration of glutaraldehyde in the disinfectant solution is above or at/below the MEC of 1.5%.

**PASS** - any shade of purple color on the indicator pad, shades of both light and dark purple on the pad are acceptable

**FAIL** – appearance of any white on the indicator pad means that the solution is ineffective and should be discarded



Diagrams are for illustration purposes only.  
Always use the color blocks on the actual bottle label to interpret results

Cidex is a registered trademark of Advanced Sterilization Products, a Johnson & Johnson company.

MetriCide is a registered trademark of Metrex, a subsidiary of Sybron Dental Specialties, Inc.

## Your Independent Choice for HLD Test Strips



Toll Free: 1.800.542.4670  
Phone: 1.574.264.3440  
Fax: 1.574.266.6222  
Website: [www.serim.com](http://www.serim.com)  
E-mail: [customerservice@serim.com](mailto:customerservice@serim.com)